



Camera Card of FUS3000 and MUS3600 needs individual memory space as mandatory working condition . Without the individual memory space , FUS or MUS software may give error 0-22,0-23 after startup . The error means camera initialization failed .

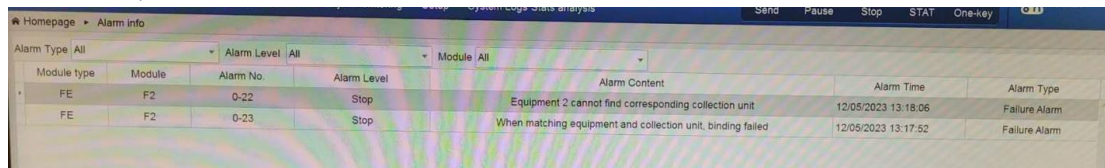


Alarm Type	Module	Alarm No.	Alarm Level	Alarm Content	Alarm Time	Alarm Type
FE	F2	0-22	Stop	Equipment 2 cannot find corresponding collection unit	12/05/2023 13:18:06	Failure Alarm
FE	F2	0-23	Stop	When matching equipment and collection unit, binding failed	12/05/2023 13:17:52	Failure Alarm

Please right click on "This PC", select properties. Check RAM ,it should lower value for useable. Such as below picture:

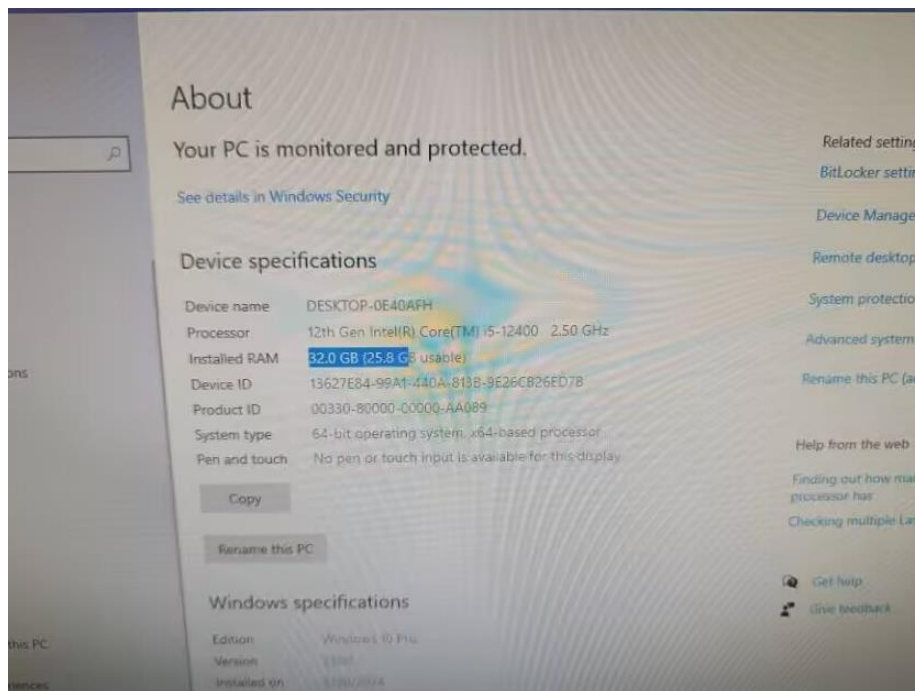


If not need to set in BIOS ,follow below steps:

1. Repwer PC, press F2 start BIOS . Click right-down conner "Advanced Mode".

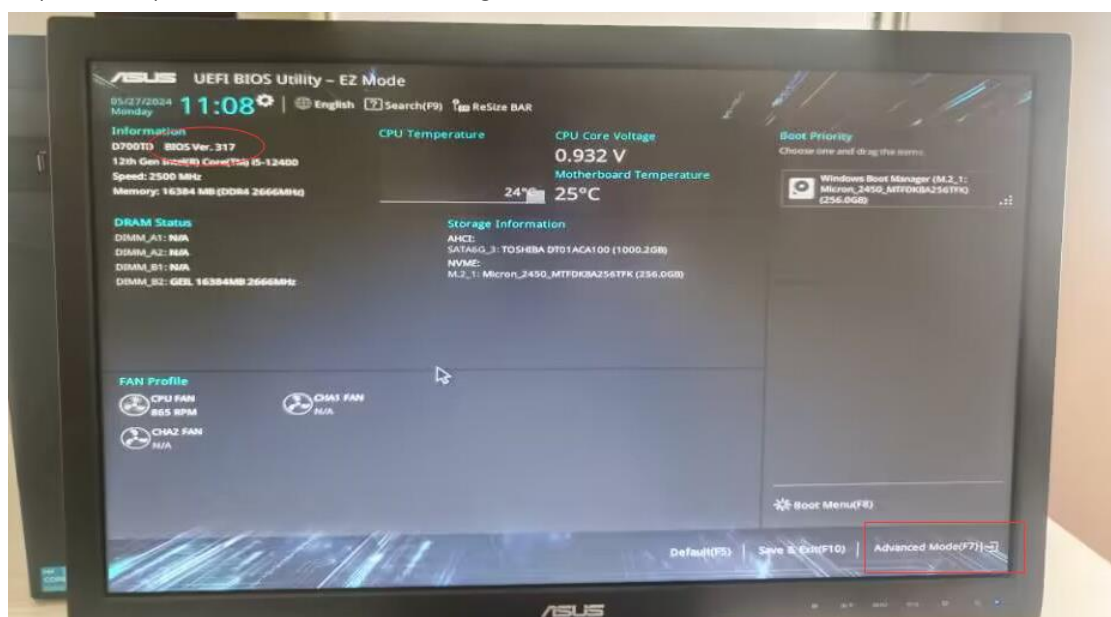


BTW,check BIOS version, if it is V317,may need to upgrade later .


2. Click Boot ,

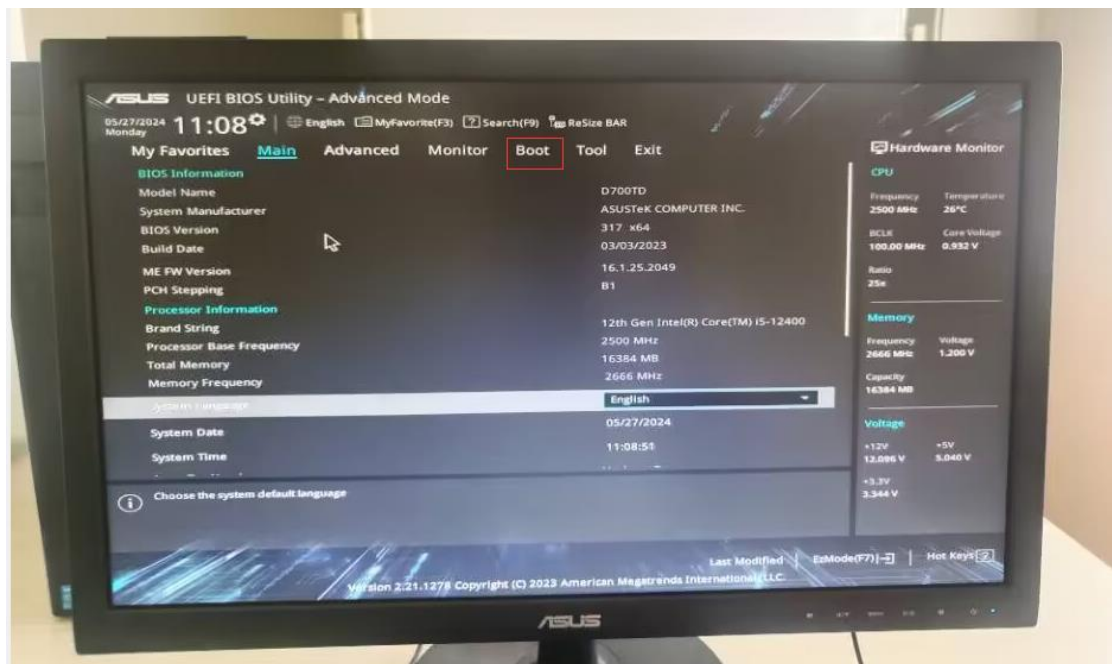


3. Secure Boot

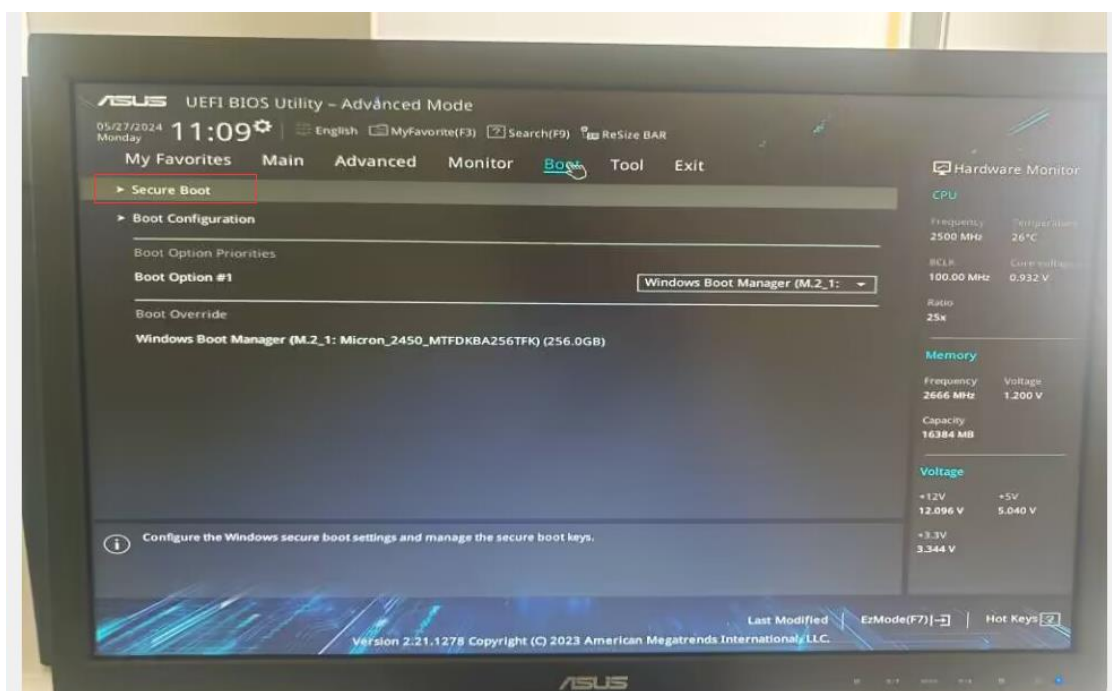


4. Change OS Type to "Other OS"

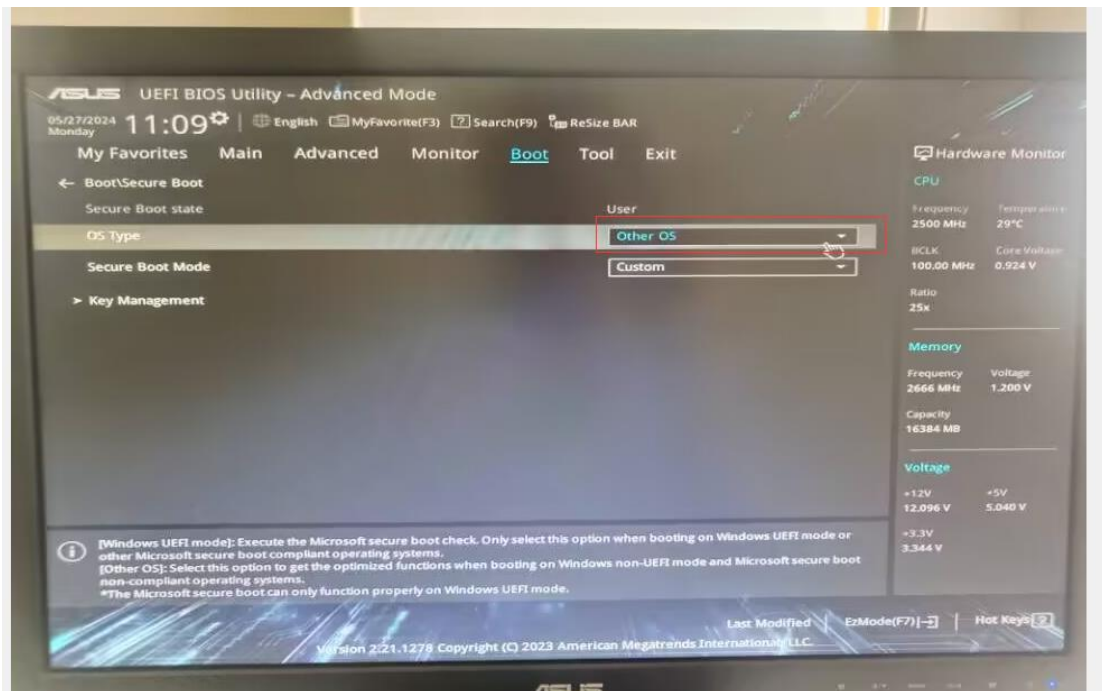


- Click Advanced-System Agent(SA) Configuration

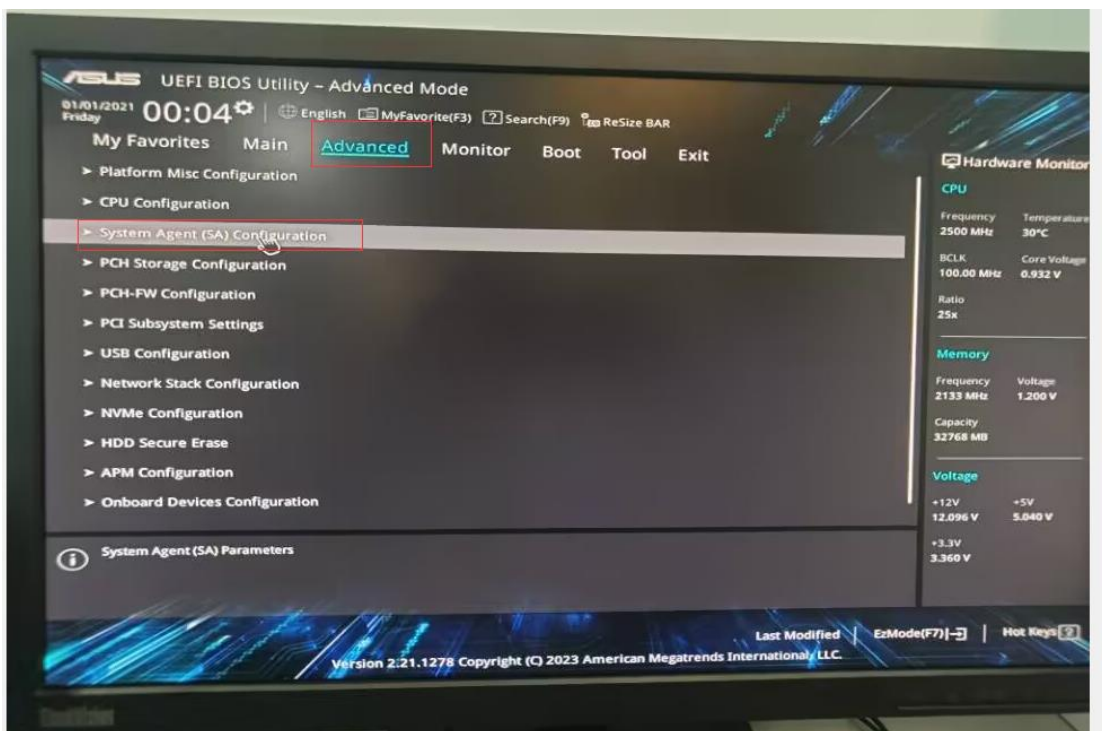

- Change VT-d to Disable.

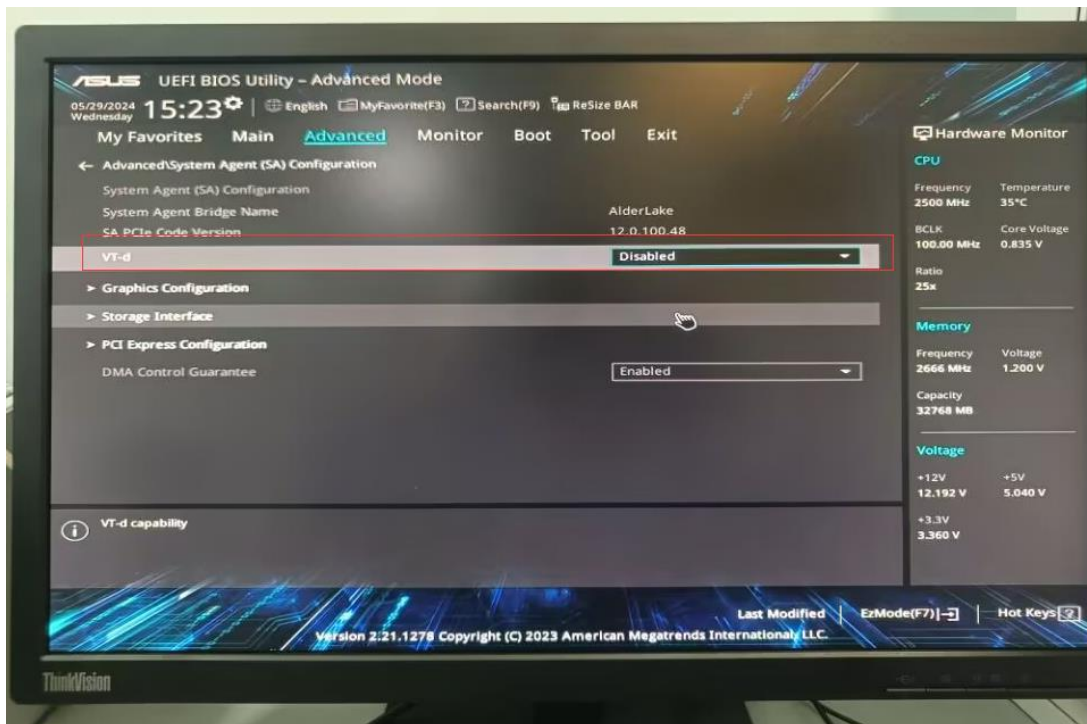

7. Press F10 ,select save and exit.

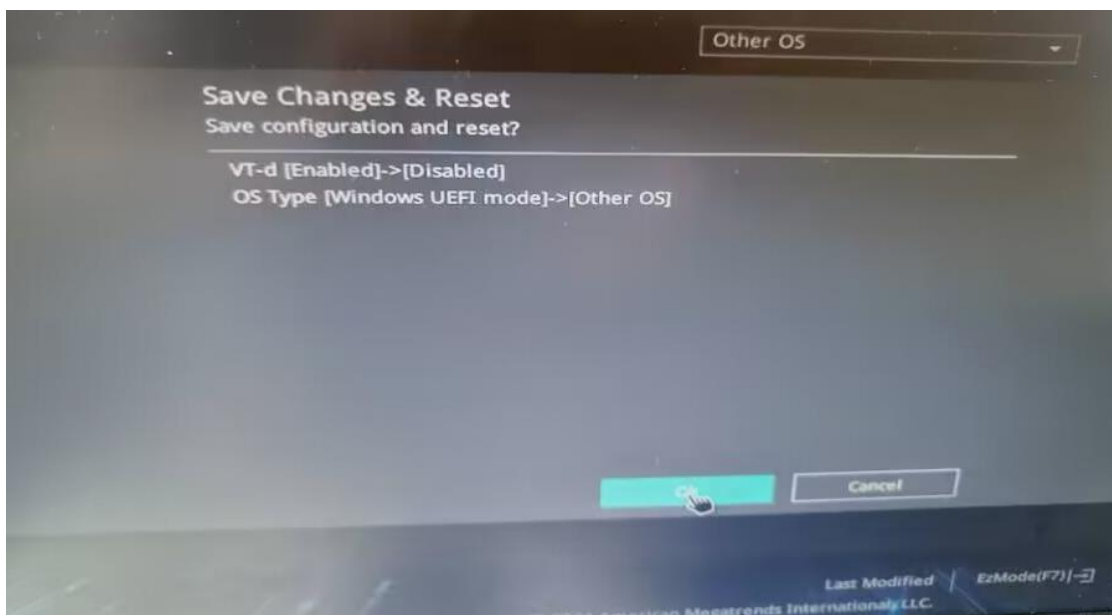


8. Till now the BIOS setting is ok. Could restart PC to cut RAM .

Please note ,if the BIOS version is V317 ,change VT-d may cant successfully. Need to upgrade BIOS version to V330.


Download the files from below link ,copy it into the PC.

https://www.asus.com.cn/displays-desktops/tower-pcs/expertcenter/expertcenter-d7-tower-d700td/helpdesk_bios/?model2Name=ExpertCenter-D7-Tower-D700TD


1. Double click it ,

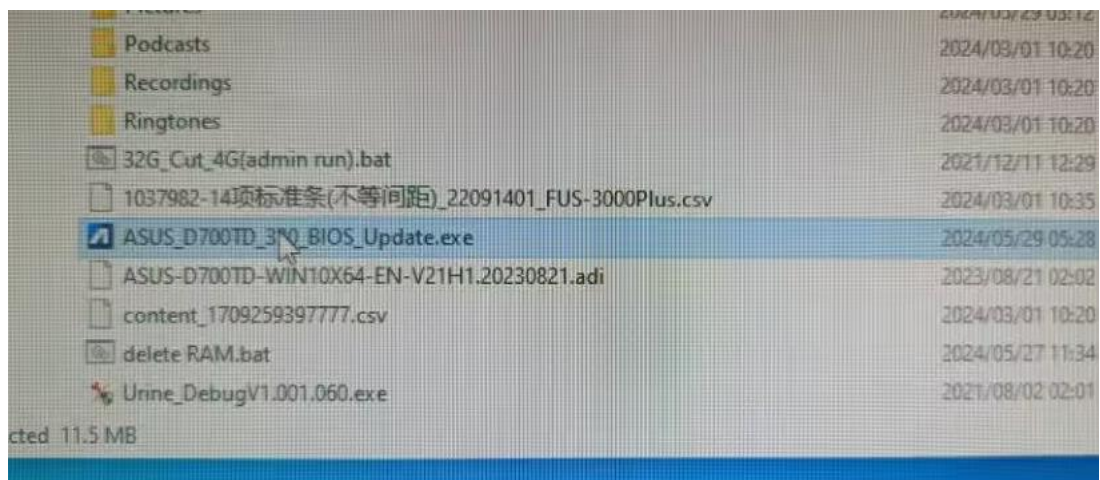


2. Run

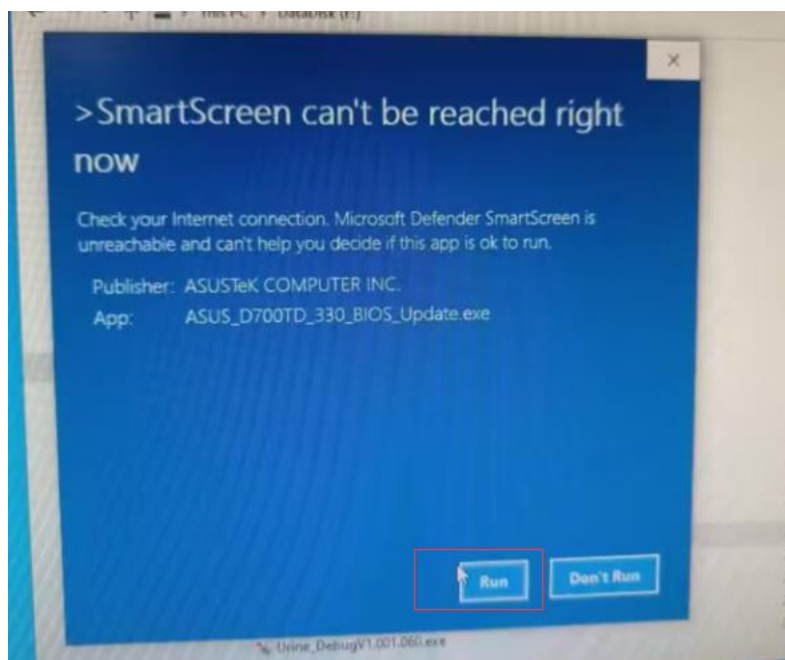


3. OK

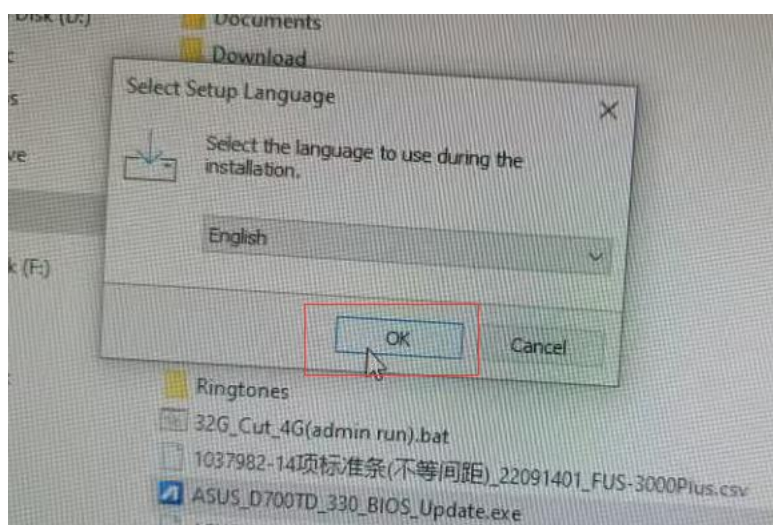


4. Next

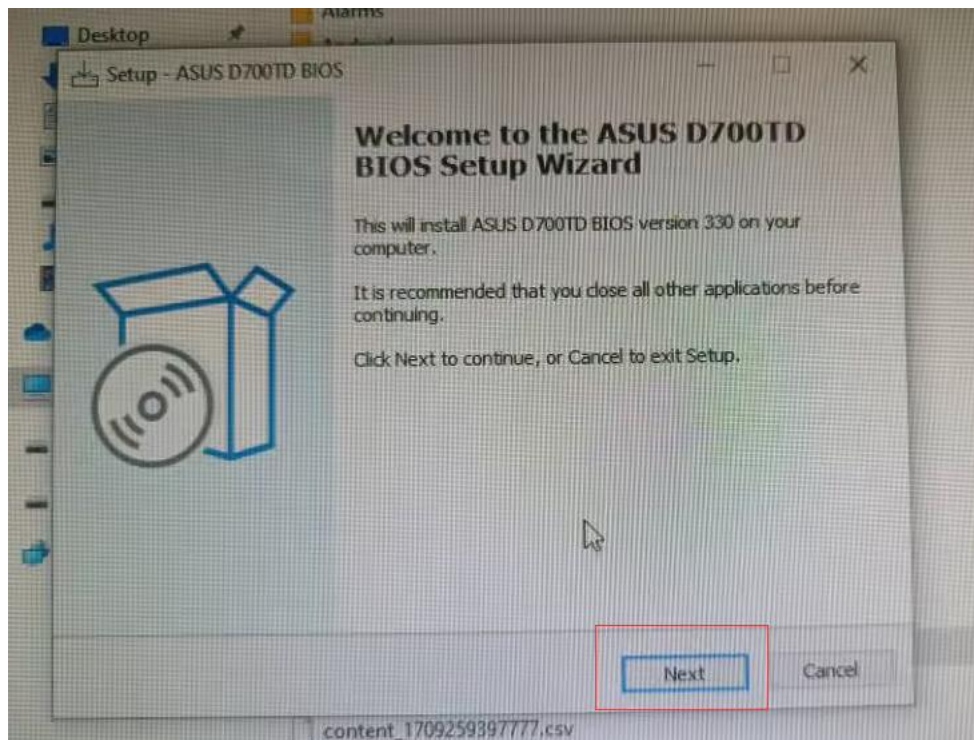


5. Accept and Next

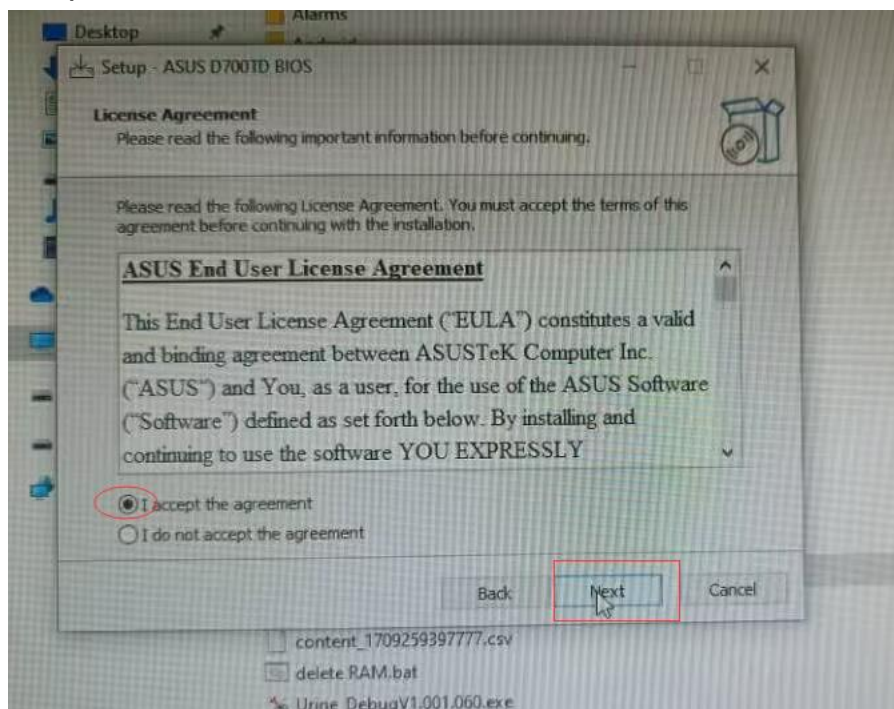


6. Next

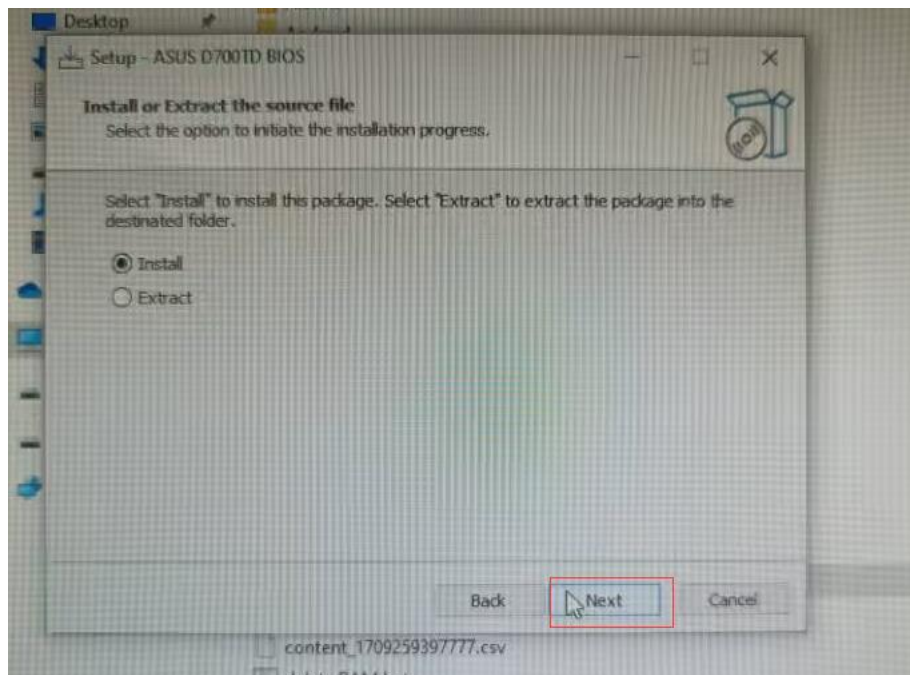


7. Next

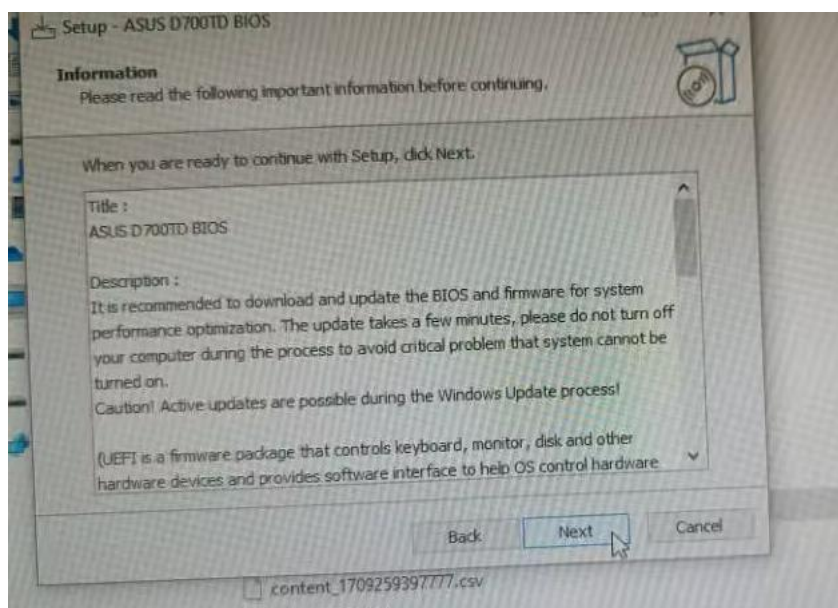


8. Install

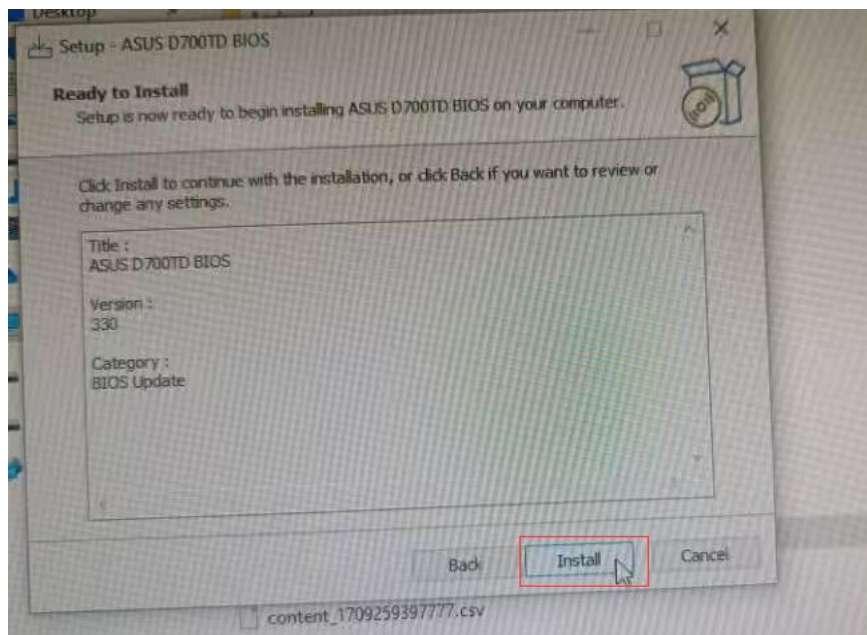

9. Finish and restart PC

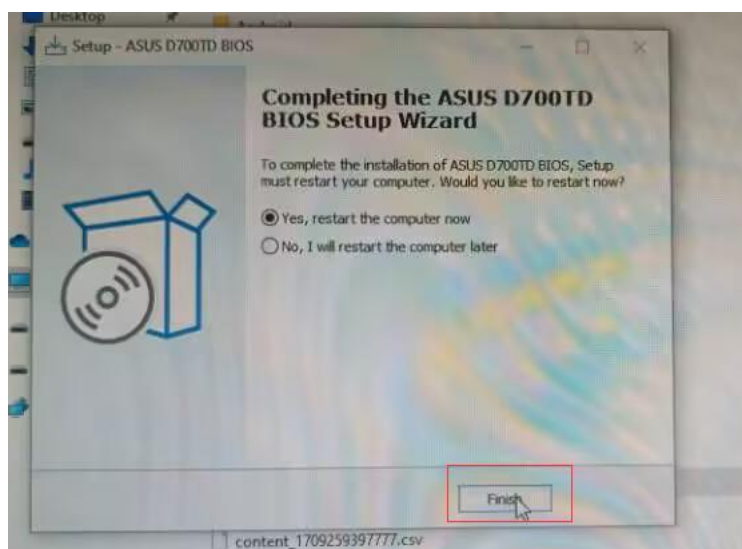

10. Wait the upgrade finish

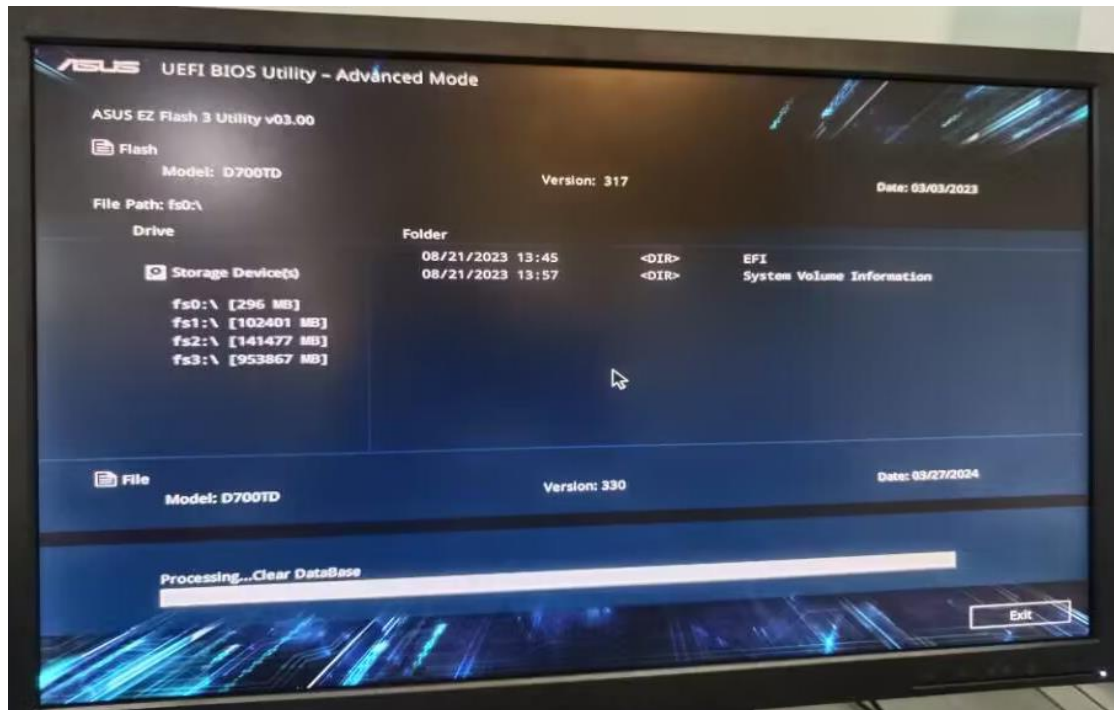

11. Double confirm OS Type and VD-t setting . Restart PC is OK.